SCHWERT
SWORD • ESPADA
EPEE • SPADA

614-30

FIG. 77R

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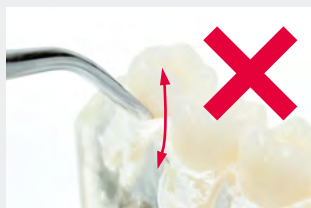
The angulation of the instrument
secures an excellent access on the
approximal contact to tooth no. 8 in
all quadrants. The fine serrated tip
allows a gentle luxation.

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The fine serrated edges assure that
the tooth will not slide from the
instrument during luxation.

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ATTENTION! Don't use the instrument
as a lever.

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ATTENTION! Please avoid any punc-
tual overstressing of the instrument
on its absolute tip, as the working tip
might break off.

We cannot assume any liability for
damages caused by improper use of
the instruments.

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The modified BECK Elevator allows an optimized interdental luxation. Standard elevators have a round back surface at the working tip. But a radius doesn't give brace support. With the flattened geometry of the new SCHWERT BECK elevator the number of supporting points is increased to 5. As a result the lifting power is more than doubled during the luxation, it is easier and time saving.

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1. The elliptical and thin tip shape simplifies the approximal insertion. The serration generates additional grip on the extracted tooth.

3. It is not foreseen, to use the instrument as an axial lever, as shown on the picture or in any other way. Because of the special shape an over strengthening of the tip may cause breakage. We assume no liability for these damage.

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2. Because of the tips shape 180° lifting power are available at the tooth to be extracted (Standard elevators are granting less than 90°). With the BECK elevator the extraction is performed much faster and easier.

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